

## QuPID™

### The visibly better one-step pregnancy test

- ◆ Easy-to-read results
- ◆ 99% accuracy and specificity
- ◆ Positive results may be seen in as little as 10 seconds\*
- ◆ Urine and serum samples



for Urine specimens



Diagnostics  
for life

# QuPID®

Everything you need in a convenient pouch (including dropper and easy-to-use instructions).

## Exceptional readability

Quick clearing background ensures easy-to-read results even with low-level specimens

Color-coded devices make it easy to differentiate between urine and serum/urine tests

## Fast

Positive results in as little as 10 seconds\*.

## Sensitive

Detects 20 mIU/mL hCG or greater in both urine and serum specimens. QuPID® Plus detects 10 mIU/mL hCG in serum

## Accurate

> 99%


## Specific

> 99%



## Test procedure

- Remove QuPID® device and disposable dropper from the foil pouch
- Label the test device with patient or control identification
- Dispense two drops of the specimen into the round sample well
- Read result as positive or negative

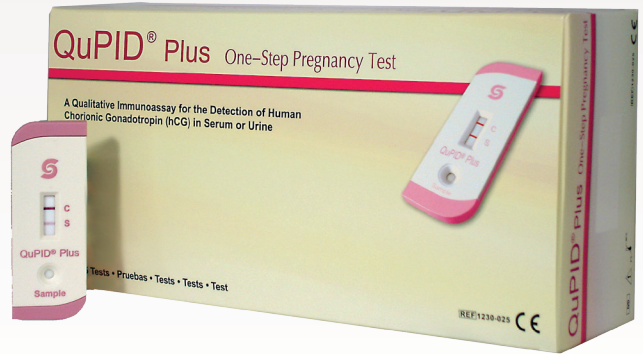

Products	Size	Reference No.	Specimen Type	Sensitivity
QuPID® hCG Pregnancy Test Set	25 test	1220-025	Urine	20 mIU/mL hCG
	50 test	1220-050	Urine	20 mIU/mL hCG
QuPID® Plus hCG Pregnancy Test Set	25 test	1230-025	Serum or Urine	20 mIU/mL hCG in urine
	50 test	1230-050	Serum or Urine	10 mIU/mL hCG in serum
hCG Bi-Level Urine Controls	2 x 5 mL	1225-205		
hCG Tri-Level Serum Controls	3 x 5 mL	1235-305		

Distributed by



for Urine specimens

ISO 13485:  
2003 Certified



Manufacturer  
Stanbio Laboratory  
1261 North Main Street, Boerne,  
Texas 78006, USA  
+1 830 249 0772  
email: sales2@ekfdiagnostics.com

www.ekfdiagnostics.com

**EKF** | Diagnostics  
for life

\* The test is positive if two colored lines appear. One colored line will appear at the Specimen Zone and one at the Control Zone.